
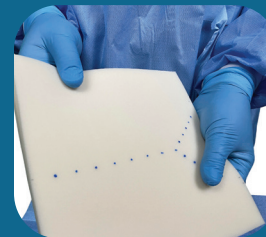


SPONGE SWICKER® Radiopaque Surgical Foam

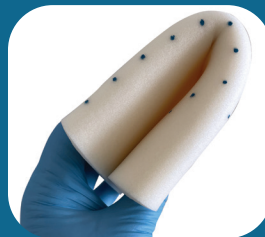

SWICKER for Surgical Procedures

Our SWICKER products are free of cellulose, one of the main causes of accidental contamination resulting in intra-operative complications, readmission, and other associated costs.

More Than Just a Sponge


- Designed for **continuous use**, highly absorbent and easy to wring clear
- Swap between one **working** sponge and one **waiting** sponge
- Reduce room **“floaters”**
- Can be used **rolled, folded, or flat**
- **Choice of size and thickness** as needed for the procedure
- Less **abrasive** means less trauma
- Use to position or **pad extremities**
- Assists in **organ isolation**
- Can be used internally in patients to help **control bleeding**
- Reduce overall **sponge count**

SWICKER Extends the Reach of Surgical Suction Devices

- Simply touching the suction tip to any part of the SWICKER will clear that area of the sponge and help manage EBL
- Allows continuous use of the SWICKER when left in position, no need to remove and replace



Reduces overall **sponge count** and biohazardous waste expenses

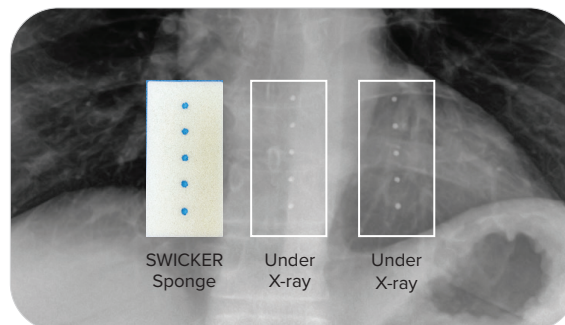



Patented biocompatible radiopaque ink dots provide exceptional visibility under X-ray [Also MRI compatible]



Used as instrument padding for stable retraction



The suction tip will clear the area of the sponge

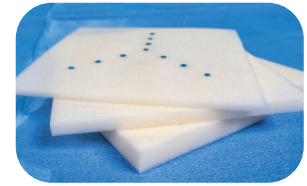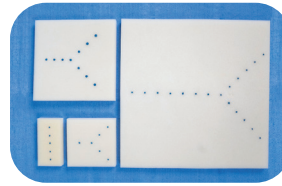

SWICKER Order Information

SWICKER Part Number	Dimensions (in)	Thickness (mm)	Firmness	Radiopaque dots per sponge
5203-I-NS	1.5 x 3	.125	Soft	5
5303-I-NS	1.5 x 3	.25	Medium	5
5403-I-NS	1.5 x 3	.50	Firm	5
5205-I-NS	3 x 3	.125	Soft	6
5305-I-NS	3 x 3	.25	Medium	6
5405-I-NS	3 x 3	.50	Firm	6
5207-I-NS	5 x 5	.125	Soft	10
5307-I-NS	5 x 5	.25	Medium	10
5407-I-NS	5 x 5	.50	Firm	10
5209-I-NS	9 x 9	.125	Soft	18
5309-I-NS	9 x 9	.25	Medium	18
5409-I-NS	9 x 9	.50	Firm	18

Remit To	Syntervention, Inc., 1444 Jeffreys Road, Suite 133 Rocky Mount, NC 27804
Contact Us	Phone: 888.505.8802 Fax: 888.853.6559 Email: info@syntervention.com
Tax ID	45-3609564
Payment Terms	Net 30 days.
Shipment	Items are shipped ground unless otherwise specified.
Product Storage	Store in a cool, dry place. Avoid exposure to moisture and excessive heat.
Hazardous Material Status	Latex, BPA, and DEHP free.
California Requirements	Meets Proposition 65 standards.
Returned Goods Policy	A Returned Goods Authorization (RGA) number is required for all returns and replacements. Authorization can be obtained by calling Syntervention Customer Service at 888.505.8802.

To view calculators visit www.syntervention.com/calculators.

To view the Learning Center Video Library visit www.syntervention.com/video-library.

Syntervention and SWICKER, are trademarks or registered trademarks of Syntervention, Inc.



SYNTERVENTION

Syntervention, Inc.
1444 Jeffreys Road, Suite 133
Rocky Mount, NC 27804
Customer Service: 888.505.8802
Fax: 888.853.6559
www.syntervention.com